

age 1 Harvesting adi pose cells for breast augmen tation



ntact lipocytes in WAL aspirates all Ranges



study: the water jet is gentle and safe in safety

hydro surgery



page 3 How to use the body jet<sup>®</sup> Ranges∄



age 3

microscope

Effect of the water-jet

viewed by the electron



nd debridem

safe and fast



**VOLUME 3** 

AquaShape® and body jet<sup>®</sup> for gentle body contouring

### WAL and Autologous Fat Transfer: A new, effective autologous fat grafting method for breast augmentation

#### Interview with Klaus Ueberreiter M.D., Asklepios Klinik Birkenwerder/Berlin by Inge Matthiesen

#### Question: What has triggered the introduction of this new fat transplantation method?

Dr. Ueberreiter: Through the publications of Sydney Coleman in New York, micro-fat grafting has been gaining popularity since the mid-90s and is now being used worldwide for the correction of tissue deficits in the facial area. It wasn't long before the use of autologous fat for breast enhancement as an alternative to silicone implants was also being discussed.

#### Question: Have advances since been made in the use of this

method for breast augmentation? Dr. Ueberreiter: In March 2007. Coleman presented the results of a study on breast augmentation with autologous fat  $(1)_i$  Zocchi and Delay, from Italy and France, published additional findings. Since then, autologous fat transfer has been generally accepted as a surgical option for breast enlargement.

#### Question: What are the disadvantages of previously used methods and the advantages of the new method you have developed?

Dr. Ueberreiter: The previously used methods of micro-fat grafting with intermediate steps such as centrifugation are very time-consuming and associated with long 

surgery times. Whereas fat cells that have been aspirated from tisusing previous methods.

vesting that requires only slightly more time for liposuction while significantly decreasing the overall procedure time.

#### Question: Can you describe the method in a little greater detail?

Dr. Ueberreiter: Fat removal with the body-jet® represents an especially gentle form of liposuction, as has since been substantiated in various publications, e.g. by Stutz, Araco and Meyer (2). We have been using this system successfully since 2004.

This success led to the idea of constructing a special collection system with additional filter function for the sterile collection of fat, an idea that was successfully implemented, thanks to the cooperation of human med AG. When the AquaShape® LipoCollector<sup>TM</sup> is used in combination with the body-jet®, the filtered, washed fat no longer has to be centrifuged and

sues using the body-iet® are already washed, and the blood content of the aspirate is very low. In our studies, we have been able to prove that this fat contains the same number of vital fat cells and fat stem cells (preadipocytes) as the fat harvested

This is the first method for fat har-

can be used for reinjection without

Another advantage lies in the fact

that, with the WAL system (water-

jet assisted liposuction), fat cells

can be harvested through very thin

lumina with openings of 0.9 mm.

As a result, the fat cells in the trans-

ferred fat tissue are more finely

distributed and can be better rein-

jected with the appropriately thin

cannulas, which reduces the risk of

Question: Are you planning to

publish the results you are achiev-

Dr. Ueberreiter: Since October

2007 we have been conducting

a prospective controlled study on

breast augmentation; the results we

have seen with the more than 30

patients who have undergone the

procedure to date are very promis-

ing. These results are being exam-

ined using various tests, including imaging procedures (MRI), both

prior to and six months after the

fat necrosis and calcification

ing with this new method?

further preparation.

where the patient cannot tolerate silicone implants in the long term and the implants must be removed. A third and very important area of application is breast reconstruction following breast cancer surgery.

#### Question: Can you tell us a little more about possible fat regeneration and the number of fat grafts that are required?

Dr. Ueberreiter: The first results are showing that an increase of breast volume of more than one cup size requires two fat grafts. For breast reconstruction following tumor resection, about four to five fat grafts are required whereby this method is much less traumatic for the patient than a free tissue transfer, for example from the abdomen.



Question: How long do the transplanted cells remain in the tissue? Dr. Ueberreiter: The integrated cells will remain in the tissue permanently. After approximately six weeks, about 80% of the transplanted cells have been integrated into the host tissue.

Question: Are there still problems associated with the masking or mimicking of possible tumor tissue in imaging procedures?

Dr. Ueberreiter: No. There have been numerous reports published on this topic since the issue was first raised.

#### Question: Dr. Ueberreiter, can you briefly summarize the advantages of this new procedure?

Dr. Ueberreiter: The advantages lie in the fact that fat can be harvested (liposuction) and transferred (fat grafting) during a single operation. The entire procedure takes only about two hours. Furthermore, the vitality and the integration rate of the fat cells harvested using this gentle procedure are very good.

The entire surgery can be performed on an outpatient basis under local anesthesia with light sedation.

(1) Sydney Coleman, A. P. Saboeiro: Sydney Coleman, A. F. Sadoerro: Fat Grafting to the Breast Revisited: Safety and Efficacy. Plast. Reconstr. Surg. 119:775, 2007.
Publications by Stutz, Araco and Meyer, please refer to pages 2 and 3 of this issue.

#### Contact

Dr. K. Ueberreiter, Asklepios Klinik Birkenwerder/Berlin, Hubertusstrasse 12-22, 16547 Birkenwerder, Germany



www.humanmed.com



Dr. Ueberreiter: The patients on whom this technique is being employed basically fall into three groups. The first group includes patients with primarily aesthetic desires for breast enlargement. Also, the procedure is very appropriate for filling defects and correcting asymmetries as well as in cases

# The influence of local anesthetics on the vitality of the pre-adipocytes

According to a study by Ueberreiter, Keck and Janke, the selection of the local anesthetic has a great impact on the vitality of the pre-adipocytes. In this investigation, human pre-adipocytes from the fat cell aspirate of 9 patients after liposuction with the body-jet<sup>®</sup> have been incubated with a variety of local anesthetics. Afterwards the vitality of the cells was tested by FACS analysis.

**Conclusion:** Only Lidocaine and Articaine/Epinephrine are applicable for the preparation of infiltration solution for liposuctions for the purpose of autologous fat harvesting.

M. Keck, J. Janke, K. Ueberreiter: Vitalitätsunter schiede von Präadipozyten unter dem Einfluß ver schiedener Lokalanästhetika. Handchir. Mikrochir Plast Chir, 2007, 215-219.

| Results               |                                       |
|-----------------------|---------------------------------------|
| Used local anesthetic | Percentage of vital<br>pre-adipocytes |
| Prilocain             | 21,7%                                 |
| Ropivacain            | 58,8%                                 |
| Articain, Epinephrin  | 65,3%                                 |
| Lidocain              | 76,5%                                 |
| Controls (NaCl)       | 92,8%                                 |

Top Five Surgical Cosmetic Procedures in 2007

# Smooth and gentle on blood and lymph vessels – intact lipocytes

In a new study by Stutz and Krahl published in Aesthetic Plastic Surgery in 2008, the authors are concluding:

"A paradigm shift has occurred with the introduction of water-jet assisted liposuction. For this method no tumescence (firm-elastic infiltration condition with high tissue pressure) is necessary. Likewise no pre-infiltration period for the homogenisation of the adipose tissue is required. The aspiration procedure is started immediately after the anaesthesia has taken effect."

"The atraumatic, anatomically appropriate procedure of water-jet assisted liposuction (WAL; body-jet®) available today represents a promising treatment for lipoedema patients who



(D2-40 antibody staining)



with strands of collagen- fibrous connection tissue

generally suffer from severe subjective and objective impairment. Liposuction treatment can bring long-term improvement if the operative technique focuses on lymph vessel preservation. Immunohistological analyses show minimal evidence of lymph vessel structures in lipoaspirates."

"The histological analysis of the aspirates documents a relatively specific removal ("apheresis") of primarily intact lipocytes with low vascular amount. In the lipoaspirates of 28 of the 30 investigated lipoaspirates (patients), the lipocytes were found to be prodiminantly (>70%) intact." "The analysis of liposuction aspirates from 60 lower extremities obtained from the inner knee area, which represents an especially high-risk region for this type of operation, showed that only minimal or no injury was done to the lymph vessels, if the liposuction procedure was performed strictly parallel to the axis of the lymph collectors.

The immunohistochemical evaluation also confirmed the assumption that a state of tumescence is not required for the water-jet assisted liposuction (WAL) procedure preserving the structural integrity of lymph vessels. It was also proven that, when the WAL technique is used, the pre-infiltration period for the tumescent fluid did not have to be observed."

J.J. Stutz, D. Krahl: Water Jet-Assisted Liposuction for Patients with Lipoedema: Histologic and Immunohistologic Analysis of the Aspirates of 30 Lipoedema Patients. Aesth Plast Surg, 2008.

-----

# Investigation on tissue effects of waterassisted liposuction with the body-jet®

In a recent investigation at the University of Greifswald, the effect of the body-jet<sup>®</sup> water spray has been studied on different tissue structures including

- skin,
- adipose tissue, adiacent muscle tissue.
- blood vessels and nerves.

All samples were exposed to the five application ranges (Range 1 to 5) of the body-jet<sup>®</sup> water jet with a 3.5 mm infiltration cannula. The duration of the saline spray application was 30 seconds within a space of one centimeter in all five ranges. This procedure represents a substantially increased impact on the tissues, as normal application times would only be up to 5 seconds and while moving the cannula. Special attention was focused on the affection of vessels and nerval structures

The researchers conclude that "no vessels or nerval structures were destroyed in the corium/epidermis or in the muscle tissue, even with deep penetration. Our findings indicate that no scarring is to be expected after the edema recedes, but a slight reactive fibrosis is possible. This would lead to a tightening of the tissue which could be seen as a firming effect."

The impact of the body-jet<sup>®</sup> water spray on vascular and nerval structures was analyzed by immunohistochemical staining. Samples representing all three specimens vessels and nerves were marked by CD34 and S100 in order to investigate them for lesions of that structures.

"In all specimens neither in the dermis nor in the subcutis or adjacent muscles a rupture or lesion of vessels or nerves was observed."

According to the findings, "a large range of action of the water jet was apparent in pure adipose tissue. Connective tissue strands in the subcutis and corium/epidermis impeded the water jet, depending on the thickness of the fibres. Multiple successive connective tissue strands effectively shortened the range of action. No destruction of connective tissue was





#### seen."

"Penetration of water into the corium and/or the epidermis would lead to edema, which would subsequently be drained via the lymph vessels. The present findings suggest that no scar formation is to be expected."

"Wherever strands of connective tissue in the subcutis do not cushion the water jet before it reached the muscle layer, thin, diffuse distribution inside and between the individual muscle fibers would occur. Here, too, edema would result from the water penetrating the muscle tissue. This water would drain over the space of a few days through the lymph vessels. However, there could be transient swelling in the muscle fibers and probably slight tenderness in the muscles for a few days."

#### Contact:

Professor M. Jünger, M.D., Department of Dermatology, Ernst-Moritz-Arndt-University, Greifswald, B. Arheilger M.D., University of Rostock, Germany

#### HYDRO SURGERY NEWS

#### WATER-JET ASSISTED LIPOSUCTION

#### Clinical Reports on the use of water-assisted lipoplasty (WAL) with the body-jet<sup>®</sup>

Water-assisted lipoplasty with the body-jet® is being described in the literature for the first time with 280 patients treated from October 1999 on.(1). Since then, the body-jet<sup>®</sup> is used for lipoplasty in Europe (e.g. in the UK, The Netherlands, Spain, Switzerland, Austria, Germany), in the USA, in Asia (e.g. Japan, Korea, Hong Kong, Singapore), in Australia, Saudi Arabia, Iran, Brazil, Venezuela, Australia and many other countries. Clinical publications and postmarket surveillance reporting do not indicate any case of a major adverse event.

#### Clinical data on the safety of WAL with the body-jet®

For WAL with the body-iet® the incidence of complications and adverse events is very low.

In a study by Taufig (1), it is reported that no infections occurred in any of the 280 patients. In one case of 280 a hematoma appeared in the medial area of the knee. In none of





the cases was there a seroma that could be clinically established. The occurrence of ecchymosis was very rare and in these few cases minimal. The previous sensitivity of the skin of the treated areas returned after 3 days to 6 weeks. (1) In a prospective randomized trial of postoperative pain by Araco et al.

comparing body-jet® WAL ("power water-assisted") and SAL (.traditional") liposuction (2), no postoperative events and no cases of hypovolemia, hemorrhages or infections were reported for WAL and SAL. However, statistical differences were found for ecchymosis measurements, bruising reduction, and pain



measurements, as well as the number of daily analgesic pills.

The authors state that the "comparison of the pain measurements showed a significant difference, with average values 4.8-fold lower for power water- assisted (WAL bodyjet®) than for traditional liposuction (SAL) (p < 0.05).Additionally, after 4 days, 87% (28/32) of patients treated with power water-assisted liposuction (body-jet®) were completely free of pain versus 3.6% (1/28) of those treated with traditional liposuction."(2)

.Ecchymosis measurements also were significantly lower for the patients who underwent power waterassisted (body-jet®) rather than traditional liposuction (p < 0.05), and the differences were significant in every postoperative measurement."(2)

Araco et al.: "Both techniques showed a dramatic bruising reduction at postoperative day 5. However, the scores were significantly lower for the power water-assisted liposuction (body jet®) in every assessment during the first 5 postoperative days."(2)

In another study by Man and Meyer (3) the authors have found in more than 800 treatments with the bodyjet® that

- "It is notable that patient safety has increased considerably, even in extensive procedures.
- There is significantly reduced pain-related impairment during

and after the procedure compared with standard tumescent technique. Patients recover quickly and return to normal daily activities rapidly

- "Considerably less intraoperative swelling allows the surgeon to realize the target result with greater precision
- "Compared with the quantity of tumescent solution used in conventional manual lipoplasty, an average of 20% to 30% was used in preinfiltration" (when using the body-jet®)
- The length of time tumescent solution remains in the tissue, as well as the resulting absorption times, are all considerably lower for the recommended infiltration solutions (when using the body- $\mathsf{jet}^\circledast)$  compared with all other tumescence-based lipoplasty techniques." (3)

These last comments may have an aspect regarding discussions on cardiotoxicity of lidocaine and other drugs after high volume infiltration , and on volume-related electrolyte shifts that may occur during traditional liposuctions techniques.

References: 1. Taufig, A. Z.: Water-Jet Assisted Liposuction. In: Liposuction - Principles and Practice. Springer 2006; 326-330.

2. Araco, A., Gravante, M.D., Araco, F., Delogu, D., Cervelli, V.: Comparison of Power Watersted and Traditional Liposuction: A Prospective Assisted and Iraditional Liposuction: A Prospective Randomized Trial of Postoperative Pain. Aesth. Plast. Surg. 31: 259 – 265, 2007. 3. Man, D., Meyer, H.: Water Jet-Assisted Lipoplasty. Aesthetic Surgery Journal, May/June 2007, 342 – 346.

How to use the five ranges (RANGE 1-5) of the body-jet®

As described in the body-jet® Instructions for Use, the application of the fan shaped water spray (saline spray) of the body-jet® can be adjusted in five ranges (RANGE 1-5), according to the plastic surgeons requirements.

RANGE 1 to 5 are all providing the safe administration of saline via the body-jet® cannulae into the adipose tissue in order to loosen and separate the fat cells during the infiltration and aspiration process

The effect of the body-jet® ranges (RANGE 1-5) on adipose tissue and on vital structures like connective and muscle tissue, blood and lymph vessels, and nerves has been investigated in various histopathological studies.

The results of these studies indicate that in the RANGES 1- 5 of the body-jet® no damage is produced to connective tissue, muscular tissue, blood and lymph vessels, and nerves

The largest range of action and the maximum loosening (swelling) effect on the fat tissues is achieved at RANGE 2 and RANGE 3, according to a study of the University of Greifswald.

#### Rationale on the selection of the body-iet® pressure RANGEs

During a lipoplasty procedure the pressure RANGE is selected by the surgeon depending on

1. the phase of the procedure (infiltration phase, aspiration phase),

2. the anatomical area that is to be treated. 3. the condition/consistency of the tissue.

Accordingly, the following recommendations on the selection of the RANGES 1-5 are provided, based on the experience of the body-jet® users (publications, study results).

- 1. Regarding the phase of the procedure: During the infiltration phase (loosening of fat tissue by infiltration of saline solution into the intercellular space) RANGE 2 or RANGE 3 is recommended because of optimum range of action and maximum loosening (swelling) effect.During the following aspiration or suction phase RANGE 1, 2, 3 or 4 are recommended depending on the anatomical area, the condition of the tissue and the desired speed of liposuction.
- 2. Regarding the anatomical area that is to be treated: In small and/or vulnerable areas (e.g. like the inner knee area or chin area), RANGE 1 or 2 would provide the best and most gentle effect. The RANGES 2, 3 and 4 are applicable for standard procedures of liposuction, e.g. of the upper/lower abdomen, flanks and thighs, according to the surgeons demands
- 3. Regarding the condition of the tissue: RANGE 5 would only be used in highly fibrous fat tissues, e.g. during secondary liposuction procedures, and RANGES 2, 3 and 4 for regular tissue conditions. The applied RANGE is increased with increased tissue rigidity.

The effect of the body-jet<sup>®</sup> water spray during infiltration/irrigation. The loosening of fat cells - viewed by the electron microscope.



IMPRESSUM

Fax

## Safe and quick wound cleaning with the new debri-jet®

### International presentation of a new wound debridement

Initial results from a study of the University Dermatology Clinic Greifswald. were presented during the "3rd Congress of the World Union of Wound Healing Societies" in Toronto and the "18. Conference of the European Wound Management Association" in Lissabon.

PAGE 4

"debri-jet® is a new, effective and hygienic debridement procedure specific for chronic wounds. Using a planar waterjet applied tangentially to the skin, wound debris is loosened quickly, completely and effectively and simultaneously suctioned off. The results show that there is no increased risk of contamination by aerosol formation and infection"

Disadvantages of previous surgical methods

- · Painfulness and requirement of general anesthesia including problems with anesthetic contraindication
- · Delineation of the necrotic margin (necrotic margins persist postoperatively or healthy tissue is being "sacrificed"
- · Bleeding that limits visibility, and pain reactions
- · Pathogens may remain in the wound and cause recurrent infections.

#### Development of the debri-jet®

The application of a pressureregulated wateriet to the necrotic area facilitates guick and effective wound debridement by substantial prevention of bleeding combined with immediate removal of debrided material. The necrotic margins are well visualized. Advantages of the debri-jet®

- Non-invasive, selective and protective surgical technique: healthy tissue is not removed.
- Better cleaning of the wound: waterjet rinsing cleans more effectively than all other surgical procedures such as scraping, cutting, shaving.
- Removal also of hardened (partially granulated) debris by softening and rinsing off.
- Hygienic removal of loosened wound debris via a closed system
- · Improved wound healing and

increased circulation using the pulsed waterjet alternating with the applied vacuum.

### Functional operation of the debri-

A planar waterjet applied tangentially to the wound surface softens and loosens wound debris and necrotic tissue quickly, completely and painlessly, and the dispersed particles can be suctioned off in the same operational step. The device contains a hand piece with a disposable flexible applicator that adjusts to the wound site (see Fig.). Integrated into the hand piece is a pressure sensor. When the applicator is placed on the wound and the opening of the hand piece sealed with a finger, a vacuum is created that is measured by the sensor. At a defined vacuum the wateriet is produced. When the applicator is removed from the wound or the opening of the hand piece is uncovered the water supply is immediately interrupted. This switching off feature in combination with the flexible applicator minimizes the formation of aerosol and thus avoids contamination and the risk of infection

Dr. G. Daeschlein, Prof. Dr. M. Jünger, Universitätshautklinik der Ernst-Moritz-Arndt Universität Greifswald Eleischmannstr.42-44, D-17489 Greifswald, Germany

### New surgical treatment option for lymphedema

#### By J.J. Stutz M.D.

For many years lymphedema was not considered surgically treatable. Only in recent years have increasing numbers of reports from Scandinavia been published that consider surgical measures to be successful in addition to the very important conservative treatment. In a limited number of cases good results can be expected using lymph node transplants. Sufficient lymph nodes must be available on the contralateral side for this procedure.

In 2002 H. Brörson (1) published seven-year results after liposuction for secondary arm lymphedema, achieving almost complete reduction of edema formation using dry liposuction. The author emphasized the improvement in mobility which in these cases could

> Kontakt: human med AG

Wilhelm-Hennemann-Straße 9 D-19061 Schwerin Phone +49 (0) 385-39570-0 +49(0)385-39570-10

info@humanmed.com www.humanmed.com

human med

Herausgeber:

Human Med AG

Human Mea AG Wilhelm-Hennemann 19061 Schwerin

ann-Straße 9

not be achieved using conservative treatment. In this context M. Földi (2) reported that 5% of 400 lymphedema patients showed no results after a four-week intensive conservative decongestive therapy with CPT (complex physical therapy) but rather continued to increase in size. Ouality of life studies with lymphedema patients, however, have recently shown that improved mobility of the extremities contributes far more to the quality of life than edema reduction alone. At this particular point the debulking procedure of liposuction comes into effect. On the basis of our extensive experience with liposuction for lipedema, we employed this technique for non-pitting lymphedema and immunohistologically processed the aspirate, finding no D2-40 antibodies to lymphatic endothelium present. This demonstrates that wateriet-assisted liposuction does not cause damage to the lymphatic vessels when done correctly. At the same time the edema could, however, be significantly reduced. The number of non-pitting lymphedema patients treated with WAL is admittedly low and the follow-up investigation period is comparatively short, nevertheless the previous results are very promising.

References: 1. Brörson H. et al.: Complete reduction of arm lymphedema with liposuction – 7 year results. Progress in Lymphology XVIII. Lymphology 2002;35 2. Földi M. Complex Decongestive Physiothe

In Casley-Smith JR; Progress in Lymphology X. University of Adelaide Press 1985, 165-167

#### Contact

J. J. Stutz M.D. ; Schwerpunktpraxis Lipödem, Thiemitztalstrasse 6, D-95131 Schwarzenbach (www.stutz-dr.com)

Simultaneous infiltration

Dr. Inge Matthiesen matthiesen@aol.com

marketing@humanmed.com

and aspiration

Redaktion:

### AquaShape<sup>®</sup> Family As shown above, water-jet assisted

liposuction (WAL) is a particularly gentle method; postoperative pain and bruises are remarkably less with WAL compared to traditional methods. Also, WAL does not require large tumescent pre-infiltration, which helps to reduce the risk of side effects while the contours of the target area remain visible.

Several new products have been developed to support this new approach. Today, the AquaShape® product family consists of the body jet®, as well as the new AquaShape® mobile and the AquaShape® LipoCollector<sup>TM</sup>.

The AquaShape® mobile\* is a small portable water-jet device. It has been designed to facilitate the entry into the WAL world, and can be used with any suitable vacuum pump. The combination of already available suction equipment with the AquaShape® mobile provides potential for savings.(\*not available in USA)

AduaShabe<sup>®</sup> LiboCollector<sup>TM</sup>

Gesamtherstellung:

maxpress pr + werbeagentur GmbH & Co. KG

www.maxpress.de





Obotritendruck

www.obotritendruck.de



